



Premature / NICU and Full-Term Evacuation Basket Instructions

INSTRUCTIONS FOR MSEB

THE MED SLED® EVAC BASKET (MSEB) can hold multiple infants and NICU Equipment.

- 1 Remove one Basket and Vest from Med Sled® Storage Device. Take Vest out of the plastic bag.
- 2 Disconnect waist buckle and put Vest on. The Vest is one-size-fits-all. A significant overlap (or non-closure) of the front Vest panel will not affect Vest performance.
- 3 To ensure proper weight distribution of the Basket, pull the back of the Vest down, making sure the "V" neck cut in the back of the Vest is well off of the back of your neck. (See Figure 1.)
- 4 Connect the waist belt (black buckle) and tighten belt as tight as possible, maintaining necessary comfort. A tight Vest belt ensures proper weight distribution of the Basket. (See Figure 2.)
Note: if the Vest "rides up" and you feel weight on your neck vs. your shoulders, pull the back of the Vest down and retighten the waist strap.
- 5 Lift Basket with tan female buckles facing you. Clip one of the female buckles into matching male buckle on Vest. (See Figure 3.)
- 6 Lift up front of Basket and fasten remaining buckles into shoulder strap male buckles ends. Adjust straps to level the Basket. (See Figure 4.)
- 7 Wrap the infants with blankets so the infants are cocooned.
• Recommend lining infant area with disposable changing pad.
- 8 Place the infant(s) into Basket
- 9 The size of the infants may require extra padding. This can be accomplished by rolling an infant blanket and laying it around the perimeter of the infant.
- 10 At this stage you can place monitors or ventilators in Basket at the side of the Basket.
• Use unused Velcro straps to secure equipment
• Use small corner loops to secure oxygen lines. (See Figure 2.)
- 11 It is recommended that the infant(s) be kept in the Basket at the surge or triage staging area. It is a safe and comfortable environment.

CLEANING AND STORAGE FOR MSEB

- 1 Basket is Decon capable. Use standard spray and cleaning materials
- 2 Put vests into Basket. Compress sides.
- 3 Store on edge into Med Sled® Storage Device.

PRODUCT WARNINGS

- Never leave patient unattended.
- Caution: This apparatus requires 1 person to operate.
- Caution: Risk of falling in the use of this product
- If Basket or Vest is torn or damaged, discontinue use.
- Warning, Basket may tilt on uneven surface.
- Warning, safety straps produce danger of strangulation.



FIG. 1 BACK View of Vest

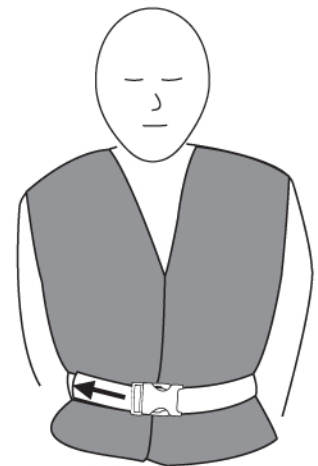


FIG. 2 Buckle Vest and pull slack to tighten waist belt

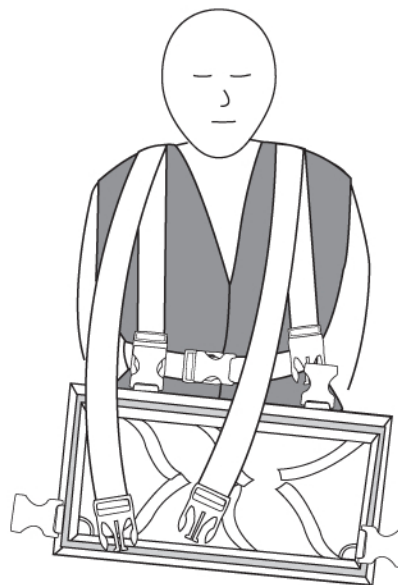


FIG. 3 Buckle Basket onto Vest. Use corner loops to secure O2 lines


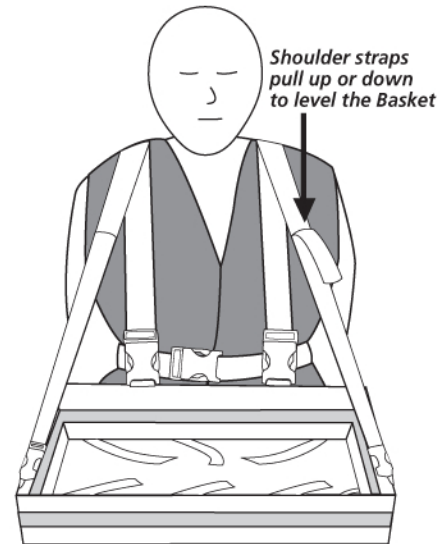


FIG. 4 Basket completely buckled and top shoulder straps adjusted so Basket is level.



NICU BABY IN MED SLED® EVAC BASKET WITH EQUIPMENT

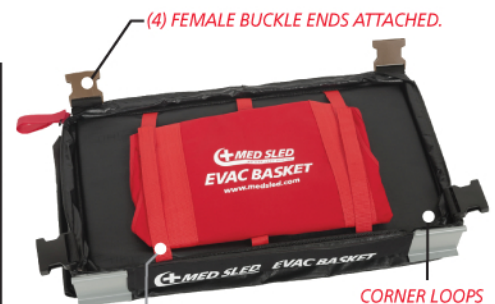

VELCRO FOR STRAPPING IN BABIES AND EQUIPMENT

CORNER LOOPS FOR SECURING O₂ LINES



BABY AND EQUIPMENT IN BASKET